

GENNY ZERO – QUICK START

1



The GENNY ZERO MD powers up in “OFF” mode: parking system disengaged, steering column at mid-height.

If the batteries aren’t installed yet:

Open the rear hatch and fit the batteries – Genny powers up on its own.

If the batteries are already installed and the unit is off:

Press and hold the “STOP” button.

2



Once the display lights up, you have two options:

hold the ON/OFF button on the cockpit display to switch the unit back off, or start it up as follows:



Level Genny out to a horizontal position.

The on-screen display updates accordingly.



Press the “PAW” button twice to lower the support legs.



Genny is now in “**PARK**” mode.

GENNY ZERO – QUICK START

3



Press and hold “PAW” to fully retract the steering column until it disappears from view.



4



Take a seat on Genny.

Note: Riders who can walk should only put weight on the footplate once seated on the cushion – not while stepping on.

GENNY ZERO – QUICK START

5



Press and hold “PAW” until the steering column extends fully and comes to a stop.

6



Press the central release button and fold the handlebar grips open.



The grips now sit level and ready for use.

7



Pressing “PAW” twice retracts the support legs again.

Genny is now in **“GUIDA”** driving mode.

GENNY ZERO – QUICK START

OPERATION

Quick-Access Menu

1



Repeatedly pressing the “STAR” button cycles through the pages of the quick-access menu.



Page 1:

Turn signals (left/right) and hazard lights can be switched on here.



Page 2:

Toggle lighting manually or set it to automatic.



Page 3:

“TURTLE” mode caps the top speed and noticeably softens steering and responsiveness.



Page 4:

Adjust the balance tilt angle to fine-tune the resting position while standing.

Notes: Above 3 km/h the menu is locked.

Turn signals can still be triggered at these speeds by swiping a finger horizontally across the display.

GENNY ZERO – QUICK START

OPERATION Parking



Slow down, come to a stop, and settle into a stable standing balance.



Press "PAW" twice to extend the support legs.

→ Genny switches to **"PARK" mode**.

Note: The steering column can now be lowered and the transfer carried out.

OPERATION Switching Off



Once no rider is on board, the ON/OFF button on the cockpit display becomes active.

To switch off:

Press the ON/OFF button

Wait a moment



Note:

Left unused, Genny switches itself off automatically after a few minutes.

GENNY ZERO – QUICK START


OPERATION Assisted Pushing “SERVOGUIDA”

1



First put Genny into “PARK” mode with the support legs down – the steering column position doesn’t matter here.

2

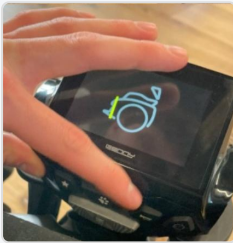

Press both together:
“STOP” **hold**
“PAW” **press twice**



The unit enters “OFF” mode – support legs retracted, steering column at mid-height.

In this state Genny can be pushed by hand with the motors off.

3



Briefly press the seat cushion.

Genny switches to “**SERVOGUIDA**” mode:

The motors engage and the unit balances itself with no rider aboard.

A light push on the steering column is enough to move it.

Note – Returning to “PARK” mode:

Press “STOP” while keeping the unit level.

Then press “PAW” twice to lower the support legs again.